

Mobi-Chair® FAQ's

How does the Mobi-Chair® beach wheelchair float?

The floating armrests and three balloon tyres enable users to float safely and comfortably. The two outside floating devices create stability in the water and ensure the chair stays upright.

Where can I use Mobi-Chair® beach wheelchair?

Mobi-Chair® can be used in calm water such as in the ocean, lakes & pools with supervision. We recommend you check the safety procedure guide before use.

How easy is it to assemble and what about transport?

Assembly takes only minutes and no tools are required. A video guide can be found on our YouTube page- <https://www.youtube.com/watch?v=U0rH74DBmjU>

How do you transport the Mobi-Chair®?

The Mobi-Chair® floating beach wheelchair frame folds down similar to an ironing board and can be transported in most vehicles.

What about maintaining the Mobi-Chair®?

After use your Mobi-Chair® floating beach wheelchair it should be thoroughly rinsed under fresh water to clean off sand, salt, chlorine and any dirt. Mobi-Chair® is rust free but if it is not maintained properly, rust will appear. Also lubricating the nuts, bolts and bearings of the wheels after every use will ensure your Mobi-Chairs longevity.

How should we store the Mobi-Chair®?

We recommend storing the Mobi-Chair® floating beach wheelchair under cover, away from rain and direct sunlight.

Is there a weight capacity of Mobi-Chair®?

The weight limit is 120kg.

Is there such a thing as a paediatric sized Mobi-Chair®?

Mobi-Chair® floating beach wheelchair comes in one-size-fits-all. Using seating spaces, the Mobi-Chair® can be adapted for most people.

How do I order Mobi-Chair® parts?

We keep all Mobi-Chair® floating beach wheelchair parts in stock. Just send us a photo of the part and we will help you to place an order.

How long will the Mobi-Chair® last?

Mobi-Chair® can last over ten years if normal use and maintenance is met.

What about Mobi-Chair® Warranty?

Push Mobility offers a one year limited guarantee from the date of purchase covering defects in materials or workmanship to the wheelchair frame only. All exceptions to this warranty are listed below. This limited one year guarantee does not include the cover seat material, armrests, or tyres. All limitations of the warranty will be assessed by Push Mobility

This one limited guarantee does not cover damage to the wheelchair frame due to:

- Normal wear and tear;
- Improper care, operation, maintenance or storage of the wheelchair;
- An accident or natural disaster;
- Modifications that have been made to the wheelchair without written consent of Push Mobility